

The CareConnect™ regulator is a non-sterile, single-patient-use device intended for use with any female external catheter, including PureWick® and PrimaFit™

Setup

1. Use standard suction tubing to attach a clean suction canister to a suction source, such as a wall regulator, suction barb, or portable suction pump.
2. Attach the CareConnect™ regulator to the patient port of the canister using the integral tubing. Note that the arrow on the label should point towards the canister.
3. Apply the external catheter to the patient per the manufacturer's recommendations.
4. Attach a suction tube to the external catheter. Any length may be used, but a 6' tube is recommended.
5. Connect the free end of the suction tube to the nipple on the CareConnect™ regulator
6. Turn the suction source to the maximum available setting i.e. "Full" or "Line".
7. Monitor the patient per the recommendations of your external catheter manufacturer.



Indications

The CareConnect™ regulator is indicated for regulating the suction to female external catheters.

Contraindications

Do not use with patients that are not indicated for your external catheter, or for applications other than collection of urine.

Disposal

After one week of use, dispose of the CareConnect™ regulator per hospital protocol.

CareConnect™ is packaged in boxes of 10.

 Boehringer Laboratories, LLC.
300 Thoms Drive, Phoenixville, PA 19460
1-800-642-4945 www.boehringerlabs.com
Made in USA

Warnings

- Rx Only** This product is for use by or on the order of a physician.
-  CareConnect™ is for single patient use. Reuse may adversely affect the performance and risks cross contamination of patients
-  CareConnect™ should be disposed of per your facility's bio-hazardous waste protocols.
-  CareConnect™ is packaged clean; but non-sterile
-  CareConnect™ must only be attached to vacuum systems. Do not attach to compressed air, nitrogen, or oxygen.
- Ensure wall outlet meets the requirements of NFPA 99. Non-compliant wall outlets may affect regulator performance.
-  Latex Free

| Part Number | Description | Ordering |
|-------------|--------------|------------------|
| 7620 | CareConnect™ | 1 Box (10 Units) |