


Installation

1. Thread the male NPT end of the Power Take-Off (PTO) into the back of the wall regulator.
2. Thread the appropriate wall fitting into the female NPT end of the PTO.
3. Mount the PTO-regulator assembly on the suction port.

Additional custom configurations are possible if using the Full Line Valve (Model 7693) . Contact Boehringer Laboratories for help.

Use

1. Connect the procedure circuit to the bubble barb end of the PTO
2. Ensure the procedure circuit has an internal regulator
3. Turn the yellow handle to FULL LINE
4. Use the procedure circuit as directed by the manufacturer.
5. The suction regulator attached to the front of the PTO can be used normally.

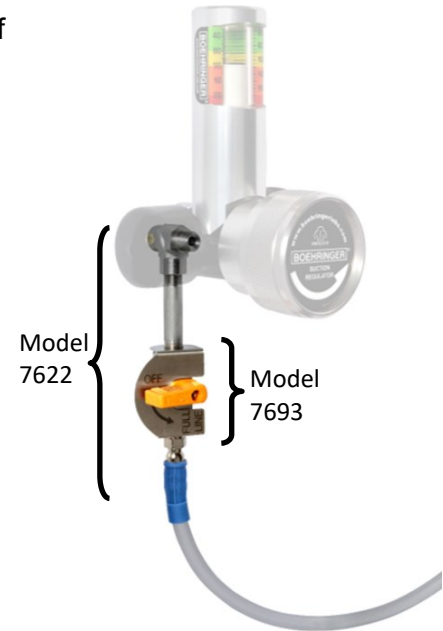

Warnings

R_x Only This product is for use by or on the order of a physician.



The PTO is packaged clean; but non-sterile

The PTO must only be attached to vacuum systems. Do not attach to compressed air, nitrogen, or oxygen.



Ensure wall outlet meets the requirements of NFPA 99. Non-compliant wall outlets may affect performance.

Do not turn on suction without a device attached

Do not apply full line suction directly to the patient or to a device without an integral suction regulator

The vacuum gauge of any regulator attached to the PTO indicates the vacuum pressure applied by the regulator only, not the PTO.

The PTO provides full line unregulated suction

Do not use in any application where full line unregulated suction is not indicated.

Do not use with any equipment that is not intended to be connected to full line unregulated suction.



Latex Free

Description

The Power Take Off provides for a second non-regulated suction port from a suction outlet that is fitted with an existing suction regulator. It is positioned between the wall suction outlet and an existing suction regulator. It includes an on/off valve. The suction provided by the Power Take Off is full line, unregulated suction only.

Indications

PTO is indicated for providing unregulated, full, line suction.



Boehringer Laboratories, LLC.
300 Thoms Drive, Phoenixville, PA 19460
1-800-642-4945 www.boehringerlabs.com
Made in USA

Part Number	Description	Ordering
7622	Power Take-Off (PTO)	1 Unit
7623	4" Power Take-Off (PTO)	1 Unit
7693	Full Line Valve	1 Unit